RESMED

Fitting Guide



- Remove the elbow from the mask by pressing the side buttons and pulling the elbow away.
- Pull headgear over your head with both lower headgear straps released.



 Adjust both the straps of the upper headgear evenly until the mask is stable.



 Bring the lower straps below your ears and loop the headgear into the lower hooks on the frame.



- Adjust the lower headgear straps evenly until the mask is stable.
- The fitting should be firm but not uncomfortable.
- Do not over-tighten.



- Connect the air tubing from your device to the elbow.
- Attach the elbow to the mask by pressing the side buttons and pushing the elbow into the mask.
- Turn on your device so it blows air.



- Undo the fastening tabs on the upper headgear straps.
- Reposition both of the straps evenly, only enough until a seal is achieved, then reattach the fastening tabs.
- Your mask is now ready to use.

Note: For additional information, refer to the Quattro Air Full Face Mask User Guide.



Quattro[™] Air